

CONNECTICUT VALLEY HOSPITAL PHYSICAL THERAPY SERVICES

EQUIPMENT MAINTENANCE AND CLEANING PROCEDURE

Flowpress Intermittent Compression System – Cleaning and Disinfecting

The following DISENFECTION procedure is completed following each patient treatment:

Requisites:

- a. Protective gloves.
- b. Cloth or paper towel.
- c. Hospital approved bleach, diluted to recommended strength in a spray bottle.

Procedure:

- a. Spray the cloth until damp and wipe the garments with hospital approved spray bleach.
- b. Air dry.

The following CLEANING procedure is completed once a week:

Requisites:

- a. Protective gloves.
- b. Cloth towel.
- c. Hospital approved bleach, diluted to recommended strength in a basin or pail.

Procedure:

- a. Wipe off the pump, tube-set, sensors, and other ancillary parts with hospital approved disinfectant.
- b. Air dry.