

# Glucometer Cleaning Instructions

## Cleaning/Disinfecting Equipment

- Perform hand hygiene before handling the meter, then don gloves.
- Use PDI Super Sani-Cloth® Germicidal Disposable Wipes to wipe down Glucometer, then allow 2 minute wet time.
- Do not allow cleaning solution to run into the meter through areas such as around the buttons or the meter's test strip or data ports.
- This will be performed after each use of a glucometer.
- Disinfection will also be performed and documented each night on third shift along with the Quality Control check.
- If the meter is not used throughout the week, this will be performed every Saturday night on third shift after Quality Control check.

