CONNECTICUT VALLEY HOSPITAL PHYSICAL THERAPY SERVICES

EQUIPMENT MAINTENANCE AND CLEANING PROCEDURE

Treadmill Cleaning

The following procedure is completed daily.

Requisites:

- a. Protective gloves.
- b. Cloth or paper towel.
- c. Hospital approved disinfectant, diluted to recommended strength in a basin or pail.

Procedure:

- a. Turn the power switch to the treadmill off.
- b. Wipe all external parts of the treadmill with a cloth or paper towel dampened with the hospital approved disinfectant diluted to the recommended strength.
- c. Pay special attention to the one-inch space on each side of the treadmill belt.
- d. Turn the power switch to the treadmill on.
- e. Start the treadmill at the lowest setting and carefully wipe the belt as it goes around, ensuring that the cloth or paper towel does not get caught on the belt.

f. Turn the belt off and reset the machine so it will be ready for the next use.